

# Morfit® – Adjustable lumbar support for drivers

## Upgrade any vehicle seat for perfect ergonomics



*I have five herniated/bulging discs in my lower back. This is the first lumbar support that really works! I'm pain free while driving long distances.*

Kate

### Guaranteed back pain relief

Morfit is an innovative lumbar support that gives immediate relief from low back pain and also works as a long-term preventative measure against future backpain. Our backs are made for standing; our spines when standing can hold a neutral curve for hours. Morfit replicates this posture while driving and offers guaranteed back pain relief.

### Medically designed

Morfit Original is Osteopath designed, patented and medical device certified. Morfit has been proven with both trials and real word testing to relieve back pain during driving.

Foam-based products are not adjustable and often far too thick to fill the narrow gap in our low back. Morfit's design is slim yet strong and supportive; seats are never one-size-fits-all but with Morfit, they can be.



### Totally adjustable

Morfit uses Interlock Foam™ in its fluid state, to move and mould into the space between the driver's back and their seat. Once shaped, a vacuum interlocks the foam particles giving a precise, custom fit and firm support that holds good posture to make any vehicle seat as supportive and comfortable as an ergonomic office chair. Once Morfit is set it remembers your shape until you choose to re-set it.

Morfit is totally adjustable, is set up quickly and easily with its strapping system and can be re-set for other vehicles, from cars, wheelchairs, through to helicopters. Morfit fits people of any shape and size and has been tested in over 200 different vehicle seats.



### How Morfit® works

- 1 Morfit starts soft and malleable.
- 2 The driver leans back against Morfit to create an exact imprint of their spinal shape.
- 3 Morfit's interlock foam is set firm and strong, using a vacuum.
- 4 Easy to get the correct support.
- 5 Can be reset as many times as you like.

## Morfit® Electric

**Colour:** Onyx black

### Pump

*Type:* Electric – fully automatic

*Function:* Program controlled

*Action:* Dual – blow and vacuum

*Operation:* Single button press

**Battery:** Rechargeable – Li-ion

**Vacuum lock valve:** Automatic

**Environment:** Indoor use only



## Morfit® Original

**Colours:** Black, Blue, Red, Yellow

### Pump

*Type:* Hand operated

*Function:* Totally manual

*Action:* Dual – blow and vacuum

*Operation:* Two hands required

**Battery:** None required

**Vacuum lock valve:** Hand operated

**Environment:** Weatherproof



## Frequently asked questions

### *Who is Morfit® for?*

Anyone who has pain or stiffness from sitting in a vehicle.

### *What type of conditions might it be good for?*

Back pain, buttock and thigh pain, disc lesions, sciatica, following back surgery.

### *Where can it be used?*

Cars, vans, trucks, motorhomes, light aircraft, helicopters, earthmovers, diggers, cranes, tractors, straddle carriers, wheelchairs, racing cars, classic cars, etc.

### *Can it be used in an office chair or on the sofa?*

Yes, but the specific shape is for vehicle seats.

*Can a passenger use it?* Yes.

### *I see it can be used in outdoor machinery, can it tolerate the elements, and can it be washed?*

Yes, it can tolerate getting wet, it does not interfere with performance and doesn't fill up with water (a little damp maybe). Morfit Electric pump is not weatherproof and recommended for indoor use only.

Yes, the cover can be washed in a washing machine.

### *Do you need further advice for back pain while driving?*

Visit the Back Pain Relief section of our website [www.morfit.co.uk](http://www.morfit.co.uk) for advice on how to prevent and reduce back pain.



**Morfit**

*Supplied by Fullmove UK Limited*

Bury St Edmunds

**T:** +44 7757 541144 **E:** [support@morfit.co.uk](mailto:support@morfit.co.uk)

[www.morfit.co.uk](http://www.morfit.co.uk)



### **Patent & Design Reg**

NZ Pat: No. 590245

Aus Pat. No. 2011101735

EC Des. Reg. 001296586-0001 & 001296586-0002

