

Introduction

Congratulations on your purchase of a YESS Range Plus sized Shower Stool or Chair from Cefndy Healthcare. Please ensure you read and understand these instructions in full before use. Please retain these for future reference.

The range is available in various configurations and sizes to suit your needs.

The plus sized shower range is designed with comfort and hygiene in mind. The seat clips off the frame to aid cleaning and the joints are fully welded to reduce bacterial entrapment. This feature also adds strength to the steel frame, which is finished in an anti-bacterial powder coat.

Available at a fixed height or with height adjustable at 25mm intervals and the feet are non-slip and non-marking. The chairs are available with soft padded vinyl upholstery or deluxe comfort PU.

For added convenience all of the shower seats are stackable.



Specifications & Pack Contents

	Mediatric Size	Bariatric	Super Bariatric
Width Between Arms	600mm	700mm	800mm
Seat Height	450 – 600mm	450 – 600mm	450 – 600mm
Seat Depth	460mm	460mm	460mm
Seat Width	580mm	680mm	680mm
Seat to Top of Arms	170mm	170mm	170mm
Max Footprint	700 x 600mm	800 x 600mm	900 x 600mm

Pack Contents

- Cefndy Shower Stool or Shower Chair
- User Instructions
- Batch Label – This will be found on the product and will include the product code as well as a number. In the event of needing to contact the manufacturer you will need to quote this number.

The Shower Seat Range

The YESS Shower seat range comes in many configurations and variations to suit your need. All the chairs come with deeply padded vinyl upholstery as standard with an option to change this for the deluxe PU foam upholstery. Where applicable chairs come with a fixed back, but also have the option of a detachable back.

The pictures below all show the Bariatric size (700mm width). They are all also available in the smaller Mediatric size (600mm width) and the larger Super-Bariatric (800mm width).



M281 - Shower Chair
M156 - Shower Stool



X281 - Shower Chair
X156 - Shower Stool



SX281 - Shower Chair
SX156 - Shower Stool

For PU upholstery add "P" to the code - For Detachable Back add "D" to the code - Add "DP" for both.

Assembly Instructions

Please note:

- When unpacking carefully examine the product to ensure it is complete.
- Before use check for any damage. If damaged please contact your supplier and refrain from using the product.
- **Do not attempt to use if the product is damaged.**



The back may have been located in an inverted position for transit (**D** versions only). If so remove the two safety pin clips located on the back to the front of the hanging brackets, slightly unscrew the hand wheels, fully withdraw the back from the brackets. Then rotate the back to an upright position sliding the back into all four brackets, insert the safety pin clips & and tighten the hand wheels.

Correct Use – Height Adjustment

- Remove the pin clip from the leg individually
- Select the height by positioning each extension leg individually aligning the holes in the extension leg and frame
- When the required height is selected refit the pin clip into the appropriate hole
- Repeat the above on the other three legs
- Ensuring all four pin clips are fully engaged and check that all four legs are set to the same height



Correct Use – Detachable Back version only

- Unscrew the handle wheels (clockwise) around half one full turn the detachable back should now slide freely in the locating brackets
- Once the desired position is reached retighten (anti-clockwise) the four handle wheels to lock the back in position
- As an added safety feature pin clips are inserted between the hanging brackets to stop the back from sliding out of the brackets **ensure these pin clips are always inserted when the product is in use**

Cleaning (decontamination)

- This product is suitable for cleaning followed by disinfection.
- Cleaning should be undertaken with a suitable detergent solution either manually or using an automated washer, always ensure detergent is rinsed then wiped of the equipment and that the equipment is dry before disinfection.
- Disinfection can be either manual or automated
 - Manual – Use a suitable disinfectant diluted to the correct concentration and wipe over the equipment. Remove the disinfectant solution with clean water and dry the equipment.
 - Automated – dependent on local processes as defined by the cleaning equipment manufacture with emphasis on achievable temperature and duration.

Note; Polyurethane Upholstery should be cleaned by manual methods only remove from the frame if using automated cleaning equipment.

Maintenance Requirements

The product should be checked on a regular basis for damage or wear and tear. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

Warning

- This product is intended for users up to a maximum of 286kg/45stone in weight.
- It is recommended that a user specific risk assessment is undertaken before use. This should include consideration for the equipment's intended use, the capacity of the user to safely use the equipment, the environment in which the equipment is intended to be used and **all** other relevant considerations.
- It is advised that the frame is checked periodically for signs of wear and tear. Any problems please report to your supplier immediately.

Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects, damage incurred through misuse is not covered. This warranty does not affect your statutory rights. Do not use in the event of fault or damage.

This product has been designed, manufactured and independently tested in the UK including cyclic testing to represent real life usages. The maximum expected product life cycle is 10 years based on the testing requirements of BS ISO 17966:2016 for static strength, impact and durability.

Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to minimise the risk of infection.
- Materials used are non-hazardous and can be disposed of in line with local recycling policies.

Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG
Tel: 0044 1745 343877
E-mail: cefndy.sales@denbighshire.gov.uk