## JWA - Air Shield Walker

### EN ANKLE/FOOT ORTHOSIS

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### ES ORTESIS DE TOBILLO/PIE

### IT ORTESI DI CAVIGLIA/PIEDE

### NL ENKEL-/VOETORTHESE

### DK ANKEL-/FODORTOSE

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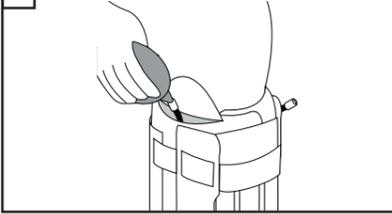
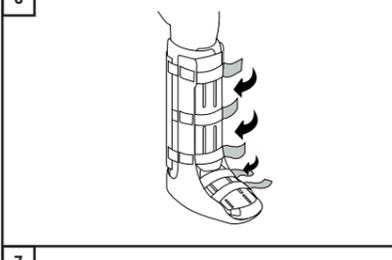
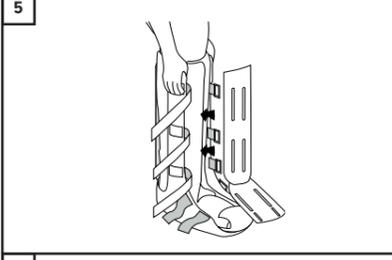
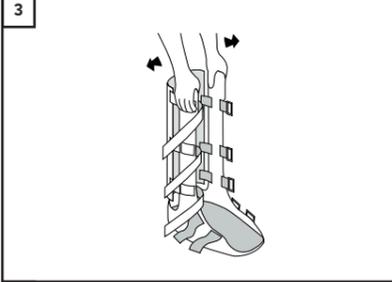
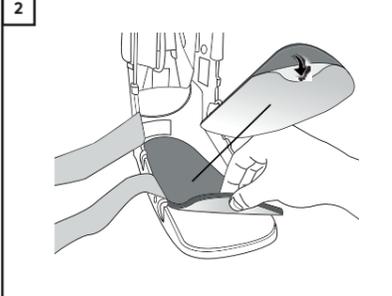
### SE ORTOS, FOT/FOTLED

### NO ORTOSE, ANKEL/FOT

### PL ORTEZA GOLEŃ/STOPA

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#### ENGLISH

#### Product Description

A lightweight low profile walker boot with independently adjustable dual airbags.

#### Usage Information (See diagram)

- When applying walker for first time open all straps and remove liner. Apply liner ensuring the heel is pushed fully into the liner. Fasten hook closures on foot and leg sections ensuring they are snug.
- The insole can be removed if you wish to insert your own custom insole or modification. To use the existing insole, please remove adhesive backing as shown and fix securely in position.
- Pull uprights apart and place foot/liner into walker ensuring heel is against posterior border of walker. Uprights should be aligned with mid ankle and lower leg.
- Apply pressure to ensure uprights are fixed to liner.
- Using hook attachments fix front plastic shield to front of liner.
- Fasten straps, from toes up, ensuring they are firm but not too tight.
- Use hand pump provided. Ensure correct side of pump is used- marked IN for inflation & OUT for deflation. Push the pump well into the valve positioned on the medial and lateral side of the walker and press the pump to inflate to the required level ensuring patient feedback is used to determine comfort.

#### Indications for Use

- Foot and ankle conditions that benefit from motion restriction.
- Stable foot fractures.
- Tendon or ligamentous sprain.
- Post-operative and rehab phase.
- Lower extremity soft tissue injuries.

#### Contraindications

- Not suitable for unstable fractures or fractures of the upper part of the tibia or fibula.
- Do not apply the product in direct contact with broken skin.

#### Warnings and Precautions

- We recommend the initial fitting of this brace be conducted by a suitably qualified healthcare professional who will advise the period of use.
- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the brace.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the brace may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the brace is not being worn or when being washed.

#### Washing and Care Instructions

- Remove liner from boot.
- Hand wash liner with mild detergent.
- Do not ring or tumble dry, leave to air dry flat.
- Wipe down the shell with clean damp cloth and mild detergent and allow to air dry.

#### Product Family Composition

Polypropylene, Polyester, PU, EVA, Nylon, PVC, Steel, POM.

#### Natural Rubber - diagram 8 only.

#### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

#### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

#### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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