## AWR - Air Walker

### EN ANKLE/FOOT ORTHOSIS

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### ES ORTESIS DE TOBILLO/PIE

### IT ORTESI DI CAVIGLIA/PIEDE

### NL ENKEL-/VOETORTHESE

### DK ANKEL-/FODORTOSE

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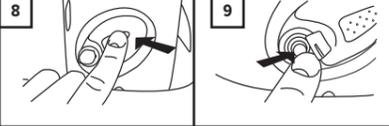
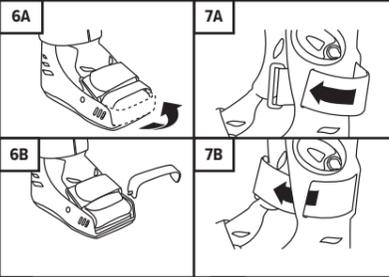
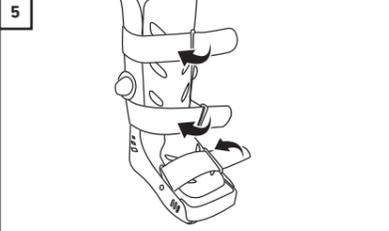
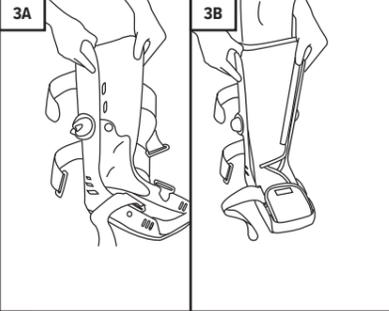
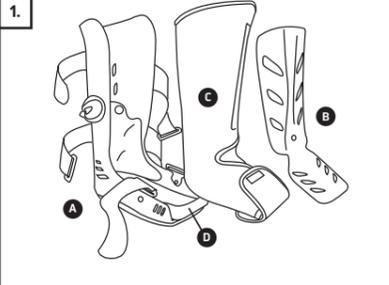
### SE ORTOS, FOT/FOTLED

### NO ORTOSE, ANKEL/FOT

### PL ORTEZA GOLEŃ/STOPA

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#### ENGLISH

##### Product Description

A walker boot with a fully integrated inflation system that allows independent adjustment.

##### Usage Information (See diagram)

- When applying walker for first time open all straps, (A) remove anterior shell (B) then remove liner (C). Toe guard (D) can be adjusted or removed (See step 7 below).
- Apply liner ensuring the heel is pushed fully into the liner. Fasten hook closures on foot and leg sections ensuring they are snug.
- Place foot/liner into walker ensuring heel is against posterior border of walker. 3b) Apply pressure to fix liner to shell
- Using hook attachments fix front plastic shell to front of liner.
- Fasten straps, from toes up, ensuring they are firm but not too tight.
- 6a & b) Toe guard can be adjusted or removed if required.
- 7a & b) If required increase compression on anterior shell by inserting both ends of ankle strap through slots in walker section as shown.
- Press the pumps to inflate air cells to the required level. Ensure patient feedback is used to determine comfort. Both pumps can be pressed simultaneously for balanced pressure.
- Open the cap and press the valve to deflate.

##### Indications for Use

- Foot and ankle conditions that benefit from motion restriction.
- Stable foot fractures.
- Tendon or ligamentous sprain.
- Postoperative and rehab phase.
- Lower extremity soft tissue injuries.

##### Contraindications

- Not suitable for unstable fractures or fractures of the upper part of the tibia or fibula.
- Do not apply the product in direct contact with broken skin.

##### Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

##### Washing and Care Instructions

- Remove liner from boot.
- Hand wash liner with mild detergent.
- Do not ring or tumble dry, leave to air dry flat.
- Wipe down the shell with clean damp cloth and mild detergent and allow to air dry.

##### Product Family Composition

ABS, EVA, Glass Fibre, Nylon, Polyamide, Polyester, Polypropylene, POM, Steel, TPR, TPU.

##### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

##### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

##### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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